

INSTRUCTION MANUAL
ALICIA PRONE STANDER
Size Large—APS3



Read and follow all instructions for assembly and use before using this product

Warnings

The use and adjustment of this stander must only be with the recommendation and guidance of a qualified therapist or physician. Adult supervision is required at all times. Never leave a child unattended while the product is in use.

Prospect Designs, Inc will not be responsible for damage or injury resulting from any modification to this equipment by anyone other than the manufacturer.

Prospect Designs, Inc will not be responsible for damage or injury resulting from misuse or neglect of this equipment.

USE

The Alicia Prone stander is designed to provide the child with ability to experience weight bearing in an upright or forward leaning position while the combination of adjustments and supports hold the child in good postural alignment.

Never leave a child unattended.

CARE

Regular inspections should be done to check for loose or missing hardware or any signs of cracked or broken parts. Contact the manufacturer immediately if you suspect an unsafe condition.

All components may be cleaned with a mild soapy water or non-abrasive commercial cleaner approved for use on vinyl and wood. Care should be taken to not use excessive amounts of water and wood parts should be wiped dry after cleaning.

WARRANTY

All Prospect Designs, Inc products are guaranteed to be free from defects in materials and workmanship for the life of the product. Should any part break or fail Prospect Designs, Inc. will repair or replace it at no charge to the original owner

FITTING AND ADJUSTING THE ALICIA PRONE STANDER APS3

Height Adjustment


The height of the Alicia Prone Stander is generally adjusted in relation to the height of the child's mid-chest height. The top of the chest pad should come up to about in line with the child's armpit or axilla .

Each child is different so this and every adjustment should be determined by a therapist or physician.

To adjust the height of the stander you need to loosen the **height adjustment knobs** on each side of the stander. The whole upper portion of the stander can then be raised or lowered to the appropriate height. Be sure to re-tighten the knobs securely.

As the stander is raised or lowered the height from the foot platform to all other components will change also and may need to be re-adjusted.

Stander at tallest height

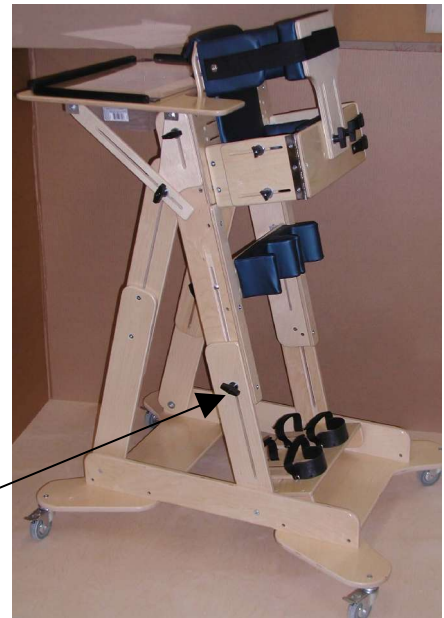

Height
adjustment
knob



Stander at lowest height


Angle Adjustment

The angle of the prone stander can be adjusted from an upright, 90° position to approximately 30° forward tilt. To change the angle of the stander loosen the two angle adjustment knobs and move the stander to the desired angle. Be sure to re-tighten the knobs firmly.

Angle
Adjustment
knobs



Stander in the upright position

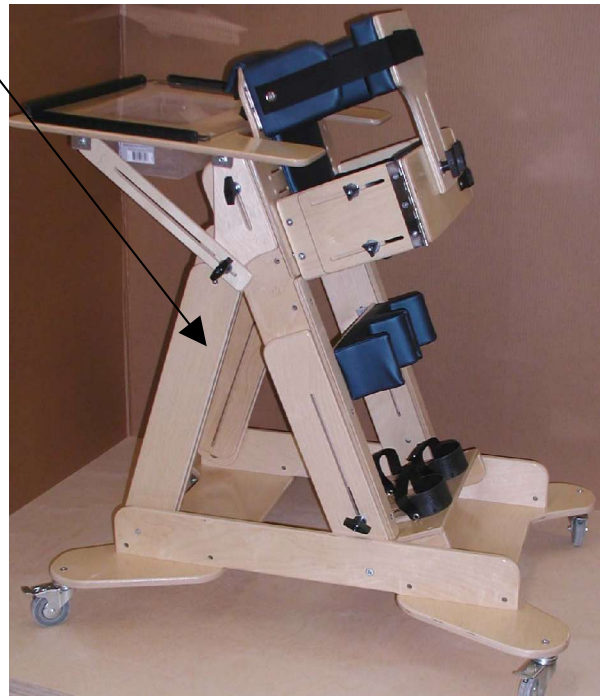

Stander with maximum forward tilt

Tray Adjustment


The tray is adjustable for both height and angle.

HEIGHT ADJUSTMENT


Loosen the knobs on each side of the stander and raise or lower the whole tray assembly to the desired height.

TRAY ANGLE

Loosen the knobs on each side of the tray assembly to change the incline of the tray.

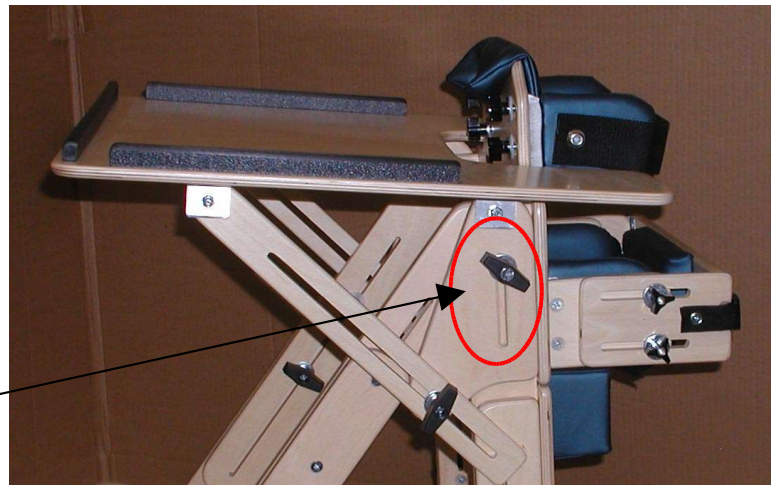

Tray in lowest position

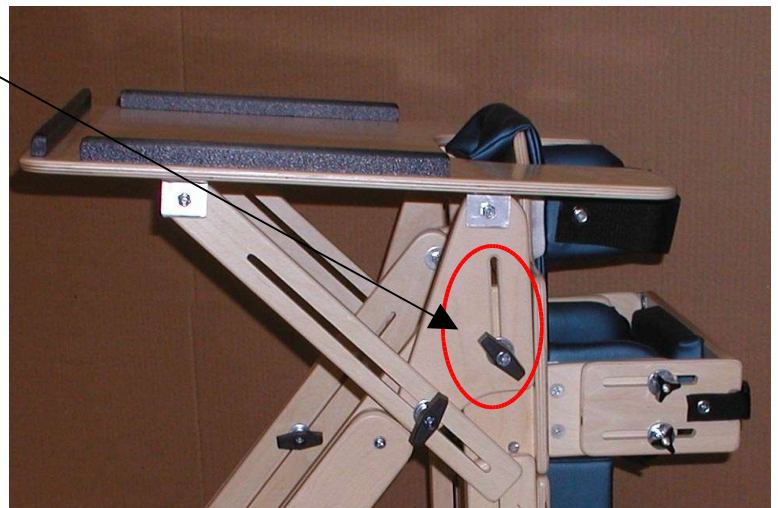

Tray in highest position

