

# INSTALLATION & SAFETY INSTRUCTIONS FOR BACK/HEADREST EXTENSION

The **BKSE1** and **BKSE2** attach to the **Posterior Pelvic Support Gate** using the two knobs (**A**) that are taped to the extension in shipping. Remove the knobs and position the back/headrest extension so that the knobs can be inserted through the vertical slots on the extension. Adjust to desired height and tighten the knobs.

When using the **BKSE1** or **BKSE2** option the velcro strap (**B**) on the upper lateral supports should be secured around the back extension once the gate has been closed to give additional support to the assembly. It is not needed for upper trunk support since that function is taken over by the back extension.

Stander pictured  
with BKSE2-  
back/headrest  
extension

